



Weck®
MemoBag™
 Specimen Retrieval System

A Bag With Memory

The Weck® MemoBag™ Specimen Retrieval System from Teleflex allows the simple deployment, handling, and retrieval of a pre-loaded tissue bag to help protect against contamination in the operating field.

Deployment is simple – the pre-loaded tissue bag is delivered through an introducer sheath via trocar into the surgical space. After deployment, a nitinol wire at the bag's upper rim "remembers" and adopts its original shape, keeping the bag open. The trocar can then be used to introduce other instruments. Specimens can be easily manipulated and removed with the bag's reliable open/close mechanism. The MemoBag Tissue Bag is also strong, able to stand up to the tension created by specimen retrieval during surgical procedures.

Distributed by:



Benefits



The Clinician
 Simple handling and deployment with a strong bag.



Your Institution
 Cost effective, no contract, with one price for all sizes.

Operative Steps*



Introduction:


Insert the MemoBag Introducer into trocar* and use the pushing cannula to deploy the MemoBag Tissue Bag into the abdomen.

*All three sizes are suited for 10 mm trocar or larger



Deployment:

Once the MemoBag Tissue Bag has been deployed, it will self-retain due to the nitinol ring at the bags upper rim.

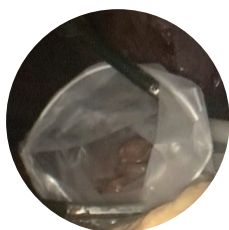

Specimen Placement:

Place specimen into the MemoBag Tissue Bag and close by pulling on the closure loop. (Note: MemoBag Tissue Bag can easily be reopened should the surgeon need to remove additional tissue).



Removal:

Once closed, remove the MemoBag Tissue Bag through the incision site.

*Always refer to the instructions for use packaged with the product for complete instructions, warnings, indications, contraindications, and precautions.

Weck® MemoBag™ Fast Facts

Made of impermeable, tear-resistant polyurethane material

Available in a range of preloaded tissue bag sizes, including a large 1200 ml capacity tissue bag

Following tissue bag deployment, trocar is free for other procedural uses

Color-coded to facilitate choice of size

Weck® MemoBag™ Specimen Retrieval System

ITEM NUMBER	SIZE/VOLUME	TROCAR	QTY/ BOX
332802-000010	200 ml	10/12 mm	5
332802-000020	800 ml	10/12 mm	5
332802-000030	1200 ml	10/12 mm	5

Distributed by:



suturexpress.com

Suture Express is a registered trademark of Suture Express, Inc.

Teleflex, the Teleflex logo and MemoBag are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this document is not a substitute for the product Instructions for Use. The products in this document may not be available in all countries. Please contact your local representative. All data current at time of printing (09/2016) Subject to technical changes without further notice.

© 2016 Teleflex Incorporated. All rights reserved. MC-002452

Teleflex®

Teleflex · 3015 Carrington Mill Boulevard, Morrisville, NC 27560
Toll Free: 866 246 6990 | Phone: +1 919 544 8000
teleflex.com